


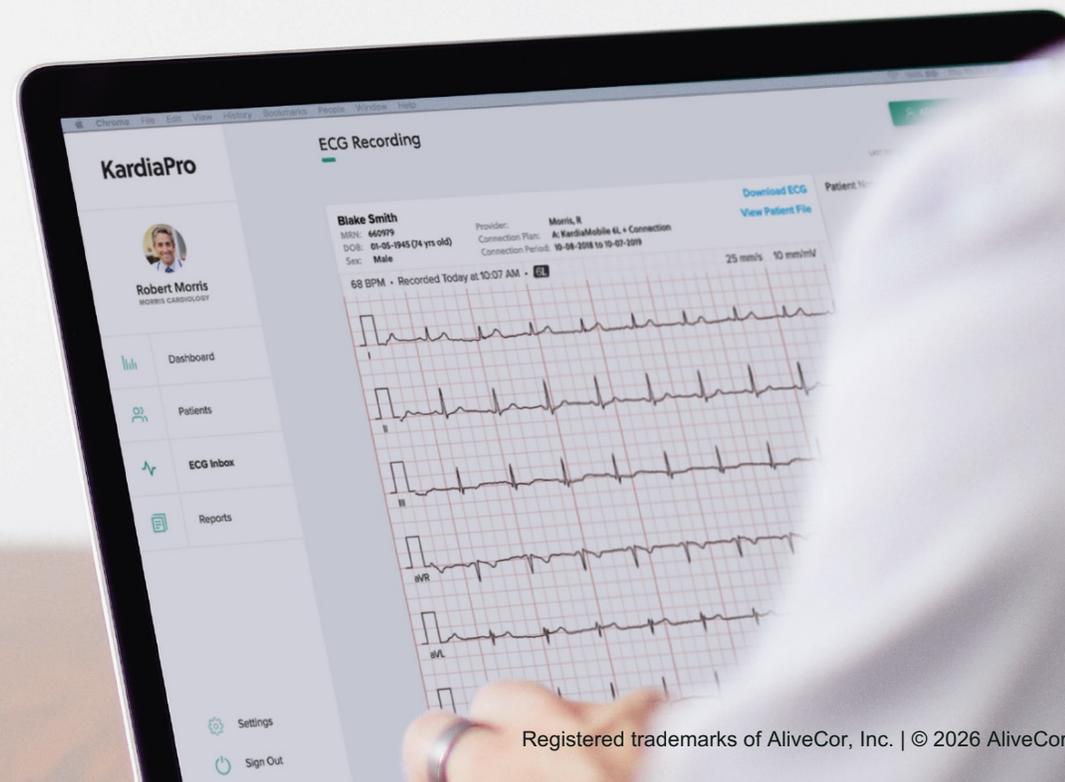

User Guide

This user guide highlights the key features of KardiaPro®, allowing your team to build a seamless workflow for Remote Patient Monitoring.



Contents

First-time Signing In Dashboard	5
Overview	6
KPro Notifications	7
Adding a Patient	11
a. Remote Monitoring	
b. In-clinic Review	
Reviewing ECGs	19
Reviewing Patient File	24
Reviewing Blood Pressure Data	34
Reviewing Reports	36
Multi-site Feature	39
User Settings	40

USE KARDIAPRO TO EASILY ORDER A KARDIA BUNDLE FOR YOUR PATIENTS:

HOW IT WORKS:

1. Practice enters patient information in KardiaPro to generate a unique referral code
2. Patient downloads the Kardia app, creates account and enters the referral code in the Kardia app
3. All patient recordings are automatically sent to the practice KardiaPro portal
4. Practice sets parameters to flag recordings you want to review
5. KardiaPro automatically generates summary reports for review


NEED ADDITIONAL SUPPORT?

Contact your AliveCor account representative directly or contact customer support at kardiaprosupport@livecor.com.

First-time signing in

Simply enter www.kardiapro.com into your web browser.

1. Select “Forgot password?”
 - Enter email address as your username.
 - You will receive an email to create a secure password to sign in to KardiaPro.

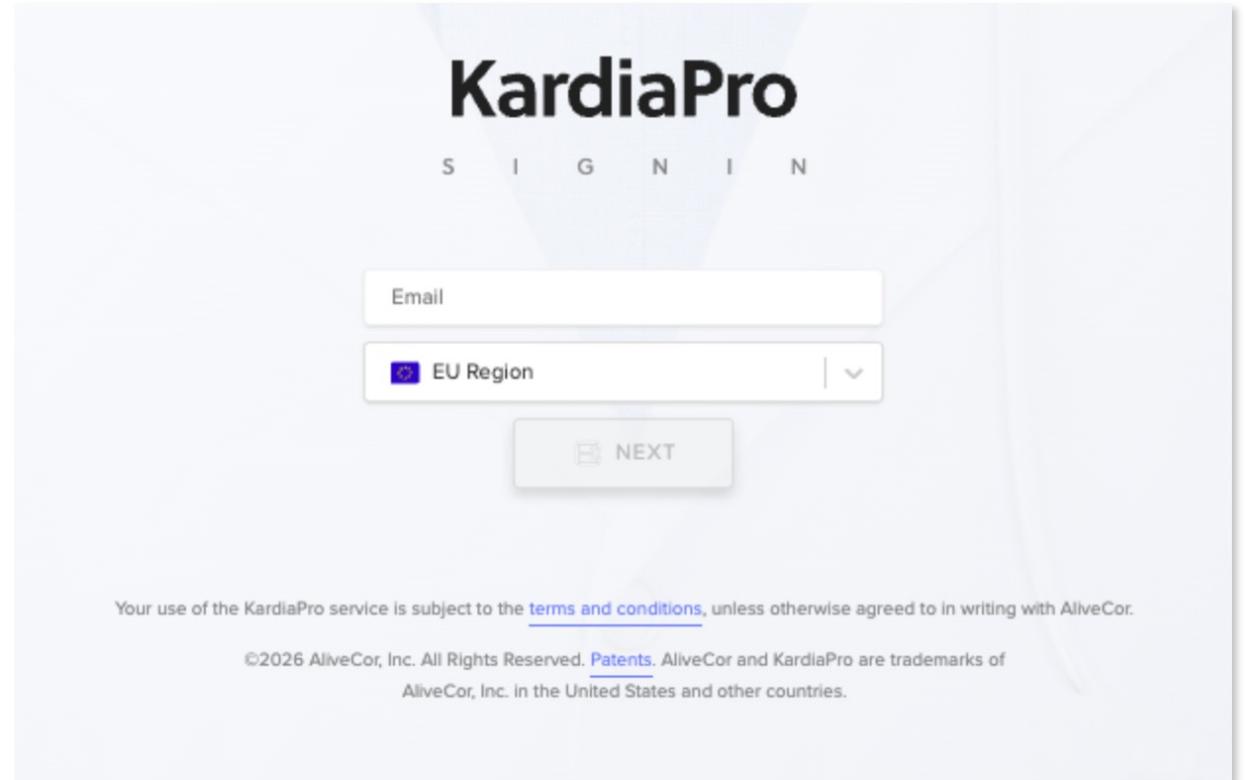

KardiaPro

S I G N I N

Email

EU Region

NEXT

Your use of the KardiaPro service is subject to the [terms and conditions](#), unless otherwise agreed to in writing with AliveCor.

©2026 AliveCor, Inc. All Rights Reserved. [Patents](#). AliveCor and KardiaPro are trademarks of AliveCor, Inc. in the United States and other countries.

KardiaPro Dashboard

This is your personal homepage to quickly access all features of KardiaPro and view new connection updates.

- 1. Patients:** Access the patient directory from here. Once in a specific patient file, select each tab to see patient information including longitudinal biometric data and summary reports.
- 2. ECG Inbox:** Access your Personal Inbox or the Practice Inbox. Find ECGs flagged for review based on the inbox settings you select for each patient.
- 3. Reports:** Summary reports automatically generated will be available for review, annotation, e-signature and export.
- 4. At a Glance:** Get a summary view of ECGs pending review and pending patient connections.
- 5. Recent ECG's:** ECGs from the last 15 days.

KardiaPro < Dashboard LAST REFRESHED 11:40 AM

Home / Fulfillment Dashboard

Dona Tomy
MORRIS CARDIOLOGY

- Dashboard
- Patients
- ECG Inbox
- Reports
- KardiaRx
- Support
- Settings
- Sign Out

At a Glance

My ECG Inbox

0 Incoming	238 Needs Review	0 Completed
------------	------------------	-------------

Practice ECG Inbox

770 Incoming	238 Needs Review	369 Completed
--------------	------------------	---------------

Patient Connections

89 Completed	7 Pending	0 15 Days Left	766 Not Ordered
--------------	-----------	----------------	-----------------

Recent ECG's

Member	Date	Time	AI Determination	HR	Email	Phone
34522577	2024-10-07	09:26:29 PM	Tachycardia	102	Arushpandya@yahoo.com	
gmtiii68@gmail.com	2024-09-28	12:30:34 AM	Normal Sinus Rhy...	68	gmtiii68@gmail.com	+15555555555
1988	2024-09-25	10:01:40 PM	Atrial flutter	63	pkuemmel477+test@alive...	

Microsoft Word

KardiaPro Instant Email Notifications

Create new notification or manage existing ones.

1. **Add Notification:** To create a new notification, go to

*Settings > Inbox Notifications
> Add Notification*

Select the Notification Type.

The screenshot displays the KardiaPro application interface. On the left is a sidebar with the user's profile (Dona Tomy, MORRIS CARDIOLOGY) and navigation menu items: Dashboard, Patients, ECG Inbox, Reports, KardiaRx, Support, Settings, and Sign Out. The main area is titled 'Inbox Notifications' and shows a list of notification settings. One notification is visible with 'Notification Type: Symptomatic' and 'Criteria: Possible Afib + HR > 120'. A toggle switch for this notification is set to 'ON'. An 'ADD NOTIFICATION' button is in the top right corner. A modal window titled 'Add Notification' is open in the center, with the text 'Use this modal to add a practice notification'. It contains a dropdown menu for 'Notification Type' and two buttons: 'CANCEL' and 'ADD NOTIFICATION'.

KardiaPro Instant Email Notifications (cont.)

Create new notification or manage existing ones.

- 2. **Filter:** Select the determination that should trigger the email notification. Only one can be added at a time.

Options include *All non-normal recordings*, *All unclassified recordings*, and specific Kardia AI determinations.

- 3. **Condition:** Set the triggering conditions for each determination. Up to five (5) conditions can be defined for each determination.

The screenshot shows the 'ADD NOTIFICATION' interface. At the top right, there is a blue button labeled 'ADD NOTIFICATION' and a status indicator 'LAST REFRESHED 11:28'. Below this, the notification type is set to 'Symptomatic' and the criteria are 'All Recordings'. A toggle switch is set to 'ON'. The 'Filter' section is highlighted with a red circle containing the number '2'. It contains a dropdown menu for 'Select Determination *' with 'All recordings' selected. Below the dropdown is a '+ ADD CONDITION' button. The 'Select recipients' section has three buttons: 'Provider' (selected), 'Account Administrator', and 'Back Office Admin'. Below these is a 'Providers *' dropdown with 'All' selected. A note below reads: 'Select a specific provider or providers otherwise all providers will be notified.' The 'Select delivery method' section has four buttons: 'Push', 'SMS', 'Email' (selected), and 'Desktop'.

This screenshot shows a detailed view of the 'Filter' section. It includes the instruction 'Select the symptom from the list below to create a filter for the notification.' and a dropdown menu for 'Select Determination *' with 'Sinus Rhythm with PVCs' selected. Below this are two conditions, each with a red circle containing the number '3' and a '- REMOVE CONDITION' link. Condition 1 has a 'Metric *' dropdown with 'HR' selected, an 'Operator *' dropdown with 'Greater Than' selected, and a 'Number *' input field with '5'. Below these are labels 'ECG Metric', 'Less Than, Greater Than, Equal Too', and 'Value'. Condition 2 has a 'Metric *' dropdown with 'QTcB (ms)' selected, an 'Operator *' dropdown with 'Less Than' selected, and a 'Number *' input field with '7'. Below these are labels 'ECG Metric', 'Less Than, Greater Than, Equal Too', and 'Value'. At the bottom is a '+ ADD CONDITION' button.

KardiaPro Instant Email Notifications (cont.)

Create new notification or manage existing ones.

4. **Select Recipient:** Specify the Provider(s) who should be notified.

5. **Send Copy To:** To send a copy of the email to recipients outside of your KardiaPro practice, check the Send copy to box and add the email address.

NOTE: Use a comma to separate multiple entries.

KardiaPro < **Inbox Notifications** ADD NOTIFICATION

LAST REFRESHED 12:04 PM

Update Password
Team Members
Invite Members
Inbox Settings
Inbox Notifications

4 **Select recipients**

Provider Account Administrator Back Office Admin

Providers*

All x

Select a specific provider or providers otherwise all providers will be notified.

Select delivery method

Push SMS **Email** Desktop

Options

If cops are to be sent to multiple email addresses, user a comma to separate.

5 Send copy to

[X] DELETE NOTIFICATION SAVE NOTIFICATION


Notification Type: Symptomatic

Criteria: **Possible Afib + HR > 120** ON

KardiaPro Instant Email Notifications (cont.)

Create new notification or manage existing ones.

- 6. **Edit Notifications:** Click on an existing Notification to edit the conditions.
- 7. **Toggle Notification On/Off:** Use the On/Off toggle to turn the Email Notification off or on.

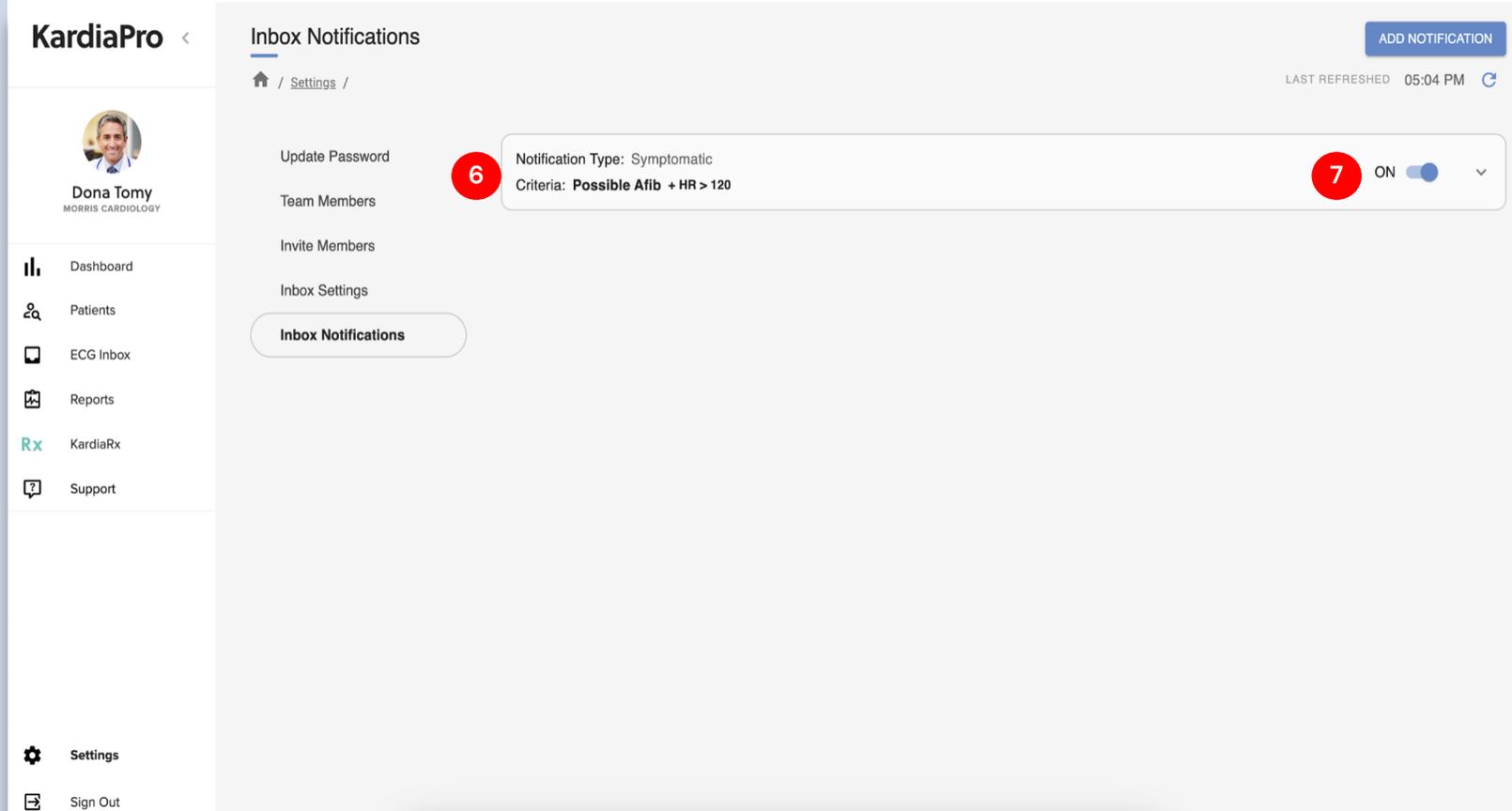

Adding a New Patient for Remote Monitoring

Adding a new patient to KardiaPro

Adding a patient is simple. Just enter patient information and select a connection plan. KardiaPro will generate a Referral Code the patient will use to purchase the Kardia Bundle you have ordered.

Patient downloads the Kardia app, creates account and enters the referral code in the Kardia app. Patient completes in-app purchase of the recommended Kardia Bundle.

1. Select the “Add Patient” button on the top right of the screen.
2. Enter patient information. The only mandatory fields are Patient Medical Record Number and Email. The email entered here will be used to send the patient an instructional email for getting started.
3. You may also enter notes about the patient here. Some practices use this field to note specific dates regarding interventions (i.e. date of ablation or cardioversion) or for entering information regarding medications (oral anticoagulation medication status, date of new blood pressure medication dose, etc.)*
4. Select “Save.”

* Depending on local information governance regulations.

KardiaPro

Add Patient

ADD PATIENT

LAST REFRESHED 02:18 PM

2 Patient Information

FIRST NAME LAST NAME

PATIENT MRN EMAIL SEX

name@example.com Select a sex

Required Required

DATE OF BIRTH ORDERING PROVIDER PHONE NUMBER

MM-DD-YYYY Unassigned XXX XXX-XXXX

3 NOTES

SAVE **4**

Adding a new patient to KardiaPro (Cont.)

STEP 1: CONNECTION PLAN

1. Select Connection Plan for the patient. This designates the length of time the patient is connect.
2. Review email address and edit as needed.

STEP 2: REPORTS

Type to select the Primary Indication ICD-10 code associated with the medical necessity for

3. Remote Patient Monitoring.
4. Select Billing Code which will then auto-generate summary reports based on the specific criteria. CPT 99091 will auto-generate a summary report every 30 days and CPT 99457 will auto-generate a report every calendar month.
5. Optional: Enter an EHR order number if required when billing Remote Patient Monitoring.

The screenshot shows the 'Add Patient' form in the KardiaPro interface. The form is divided into three main sections: 'Connection Plan', 'Reports & Billing', and 'Inbox Settings'. Each section has a 'STEP' indicator (1, 2, and 3 respectively) and a red circle with a number (1-8) pointing to specific fields. The 'Connection Plan' section (STEP 1) includes a dropdown for 'Select a Connection Plan' (1) and a text input for 'Enter Patient Email Address' (2). The 'Reports & Billing' section (STEP 2) includes dropdowns for 'Select ICD-10 Code' (3) and 'Select Billing Code' (4), and a text input for 'Enter EHR Order Number' (5). The 'Inbox Settings' section (STEP 3) includes dropdowns for 'Which Recordings to Flag' (6) and 'Flagged Recordings Go To' (7). At the bottom right, there is an 'ORDER' button (8). The interface also shows a user profile for Robert Morris, a navigation menu with options like Dashboard, Patients, ECG Inbox, Reports, Settings, and Sign Out, and a top navigation bar with 'Add Patient', 'Download Report', and 'Last Refreshed' information.

Adding a new patient to KardiaPro (Cont.)

STEP 3: INBOX SETTINGS

6. Select a filter for which recordings you want flagged for review in the ECG Inbox: all non-normal recordings, all recordings, no recordings or custom. All recordings will be available in the patient file regardless of which recordings you elect to flag for review.
7. Select an Inbox where you would like flagged recordings to be routed for review: Practice Inbox or unique Provider Inbox.
8. Select "Order."

The screenshot shows the 'Add Patient' form in the KardiaPro interface. The form is divided into three main sections: 'Connection Plan', 'Reports & Billing', and 'Inbox Settings'. Each section has a corresponding step indicator (STEP 1, STEP 2, STEP 3) on the left. The 'Connection Plan' section (STEP 1) includes a dropdown menu for 'Select a Connection Plan' (callout 1) and a text input for 'Enter Patient Email Address' (callout 2). The 'Reports & Billing' section (STEP 2) includes a dropdown for 'Select ICD-10 Code' (callout 3), a dropdown for 'Select Billing Code' (callout 4), and a text input for 'Enter EHR Order Number' (callout 5). The 'Inbox Settings' section (STEP 3) includes a dropdown for 'Which Recordings to Flag' (callout 6) and a dropdown for 'Flagged Recordings Go To' (callout 7). At the bottom of the form, there is a button labeled 'ORDER' with a checkmark icon (callout 8). The top of the form has a 'DOWNLOAD REPORT' button and an 'ADD PATIENT' button. The left sidebar shows the user profile for Robert Morris, Morris Cardiology, and navigation options: Dashboard, Patients, ECG Inbox, Reports, Settings, and Sign Out. The top navigation bar includes tabs for Order, Profile, Reports, ECG, Blood Pressure, and Weight, along with a 'LAST REFRESHED 10:38 AM' indicator.

Adding a new patient to KardiaPro (Cont.)

1. Upon completion of ordering the Kardia Bundle, the patient file will open. You will see the “pending” connection status and a 12–letter referral code for the patient to use when creating their Kardia account on their phone.
 - Provide patient with support materials.
 - The patient then downloads the Kardia app, selects “create account” and enters the 12–letter referral code when prompted. The patient will be connected to KardiaPro.

The screenshot displays the KardiaPro interface for a patient named Barbara Smith. The top navigation bar includes the KardiaPro logo, patient name, age (70 yrs old), MRN (979646), and buttons for 'DOWNLOAD REPORT' and 'ADD PATIENT'. Below the navigation bar, there are tabs for 'Order', 'Profile', 'Reports', 'ECG', 'Blood Pressure', and 'Weight'. The main content area is divided into several sections:

- Connection Status:** Shows a 'PENDING' status with a red 'R' icon. A message states: "A Kardia order was sent to this patient Today at 2:22 pm, the patient has not connected to your practice using the below code." A red circle with the number '1' is next to the referral code: **REFERRAL CODE: ABCD-EFGH-IJKL**.
- Order Information:** Contains a table with the following details:

CONNECTION PLAN	DURATION
KardiaMobile 6L + BP Cuff + 1 Year Connection	11/1/18 - 10/31/19
- Primary Indication:** Paroxysmal atrial fibrillation
- Billing Code:** CPT 99091
- EHR Order Number:** 00224423
- Which Recordings to Flag:** All non-normal recordings
- Flagged Recordings Go To:** Practice Inbox

A 'SAVE' button is located at the bottom right of the page.

Adding a new patient to KardiaPro (Cont.)

- For practices providing patients with in-office set-up and education (see support materials), use a KardiaMobile 6L demo unit to have the patient take their first ECG recording using their Kardia App. Use this opportunity to educate patients on how to take a quality ECG recording (see support materials).
- For patients monitoring blood pressure, AliveCor recommends the Omron Evolv Blood Pressure demo monitor for education. Patients can trial the demo monitor in the office, then manually enter their blood pressure recording into their Kardia app. If patient is ordering an Omron Evolv Blood Pressure monitor, they will pair their unit using Bluetooth to the Kardia app when they receive the device.

Introduction

Welcome! Your provider(s) have requested that you record ECGs from an AliveCor KardiaMobile 6L device. During this time, they will monitor your cardiac activity while you're at home or on-the-go.

You will receive an email from support@alivecor.com with a connection code, or the connection code will be sent directly from your healthcare professional to initiate the monitoring. The email, and this brochure will guide you through the steps required to take ECGs.

Getting Started

Before you begin, make sure you have the following items:

- Valid email address for health care professional to send an email with connection code.
- KardiaMobile 6L ECG device – if not offered by provider, purchase one from store.alivecor.co.uk/products/kardimobile6l or other retailers.
- Compatible phone with connection to Internet—see kardia.com/compatibility.
- Passwords to (a) unlock phone and (b) download and install the Kardia application (app).
- Kardia app installed on your mobile device – scan the appropriate QR code above to download the app.



Steps for Cardiac Monitoring

Create a Kardia Account and Link it With Your Clinician's Practice



STEP 1

Launch the Kardia app and tap **HAVE A CODE?** At the bottom of the screen.



STEP 2

Tap the first box and enter the 12-character code that you received via email or from your clinician. Next, select **SUBMIT**.



STEP 3

Select the checkboxes and tap on **NEXT** when it turns green.

Adding a new patient to KardiaPro (Cont.)

- For practices sending the patient home for set-up, refer the patients to the instructional email from KardiaPro. The email includes the 12-letter referral code. You may also elect to write down the 12-letter.



STEP 4

Enter information and tap **CREATE ACCOUNT**.



STEP 5

Enter information and tap **COMPLETE PROFILE**.



STEP 6

Tap **OKAY** once your Kardia account has been connected.

Link Your Existing Kardia Account to Your Clinician's Practice



STEP 1 – Open the Kardia app and tap the Profile button at the bottom right-hand corner of the app menu.



STEP 2 – Tap **HAVE A CODE?** Enter the code you received via email or from your clinician.

Record Your ECG*

STEP 1 – Make sure you have Bluetooth enabled on your phone.

STEP 2 – Open the app and tap **RECORD YOUR ECG**.

STEP 3 – Select **Six-Lead ECG**.

STEP 4 – As shown on page 1.

STEP 5 – With your fingers resting on the top two electrodes, place the bottom electrode of the ECG device to bare skin of your leg (inside of left ankle or above left knee). The bottom electrode should contact the skin.

STEP 6 – Hold still as your recording smooths out. Remain still as the timer counts down from 30 seconds until your ECG recording is complete.

STEP 7 – The ECG will be saved to your phone, and queued to send to your provider when an Internet connection is present.

*You can also view an online tutorial in the Kardia app.

Adding a New Patient for In-clinic Review

Adding a new patient to KardiaPro for in-clinic review

1. Select the “Add Patient” button on the top right of the screen.
2. Enter patient information. The only mandatory fields are Patient Medical Record Number and Email. The email entered here will be used to send the patient an instructional email for getting started.
3. You may also enter notes about the patient here. Some practices use this field to note specific dates regarding interventions (i.e. date of ablation or cardioversion) or for entering information regarding medications (oral anticoagulation medication status, date of new blood pressure medication dose, etc.)*
4. Select “Save.”

* Depending on local information governance regulations.

KardiaPro

Add Patient

1 ADD PATIENT

LAST REFRESHED 02:18 PM

2 Patient Information

FIRST NAME LAST NAME

PATIENT MRN EMAIL SEX

name@example.com Select a sex

Required Required

DATE OF BIRTH ORDERING PROVIDER PHONE NUMBER

MM-DD-YYYY Unassigned XXXX XXX-XXXX

3 NOTES

4 SAVE

For in-clinic monitoring, no connection or billing codes are required.

1. Select default values for options 1–5.
2. Select an Inbox where you would like flagged recordings to be routed for review: Practice Inbox or unique Provider Inbox.
3. Click “Order

The screenshot shows the 'Add Patient' interface in KardiaPro. The left sidebar displays the user profile for Robert Morris, MORRIS CARDIOLOGY, and navigation options: Dashboard, Patients, ECG Inbox, Reports, Settings, and Sign Out. The main content area is titled 'Add Patient' and includes a 'DOWNLOAD REPORT' button and an 'ADD PATIENT' button. Below the title are tabs for 'Order', 'Profile', 'Reports', 'ECG', 'Blood Pressure', and 'Weight'. The 'Order' tab is active, showing a three-step process:

- STEP 1: Connection Plan** (SELECT A CONNECTION PLAN FOR THE PATIENT)
 - 1. Select a Connection Plan (Required)
 - 2. Enter Patient Email Address (Required)
- STEP 2: Reports & Billing** (DESIGNATE BILLING CODES TO GENERATE REPORTS)
 - 3. Select ICD-10 Code
 - 4. Select Billing Code
 - 5. Enter EHR Order Number
- STEP 3: Inbox Settings** (FLAG PATIENT RECORDINGS FOR REVIEW)
 - 6. Which Recordings to Flag
 - 7. Flagged Recordings Go To

At the bottom right, there is an 'ORDER' button with a checkmark icon, labeled with callout 8.

Reviewing ECGs

When patients are added to KardiaPro, inbox settings can be configured to determine:

- Which ECGs should be flagged for review: all non-normal recordings, all recordings, no recordings or custom.
- Which inbox the flagged ECGs will be routed to: the unique Personal Inbox or Provider Inbox.

1. Select “ECG Inbox” to view ECGs flagged for review. Note: Personnel designated with as an Account Administrator can set email notifications, delivered either daily or weekly, if there are ECGs awaiting review in either the Practice Inbox or your Personal Inbox.
2. Navigate to each inbox by selecting “Personal Inbox” or “Practice Inbox.” The green circle with the numeric value designates the number of ECGs awaiting review in each inbox.
3. Archived is used for sites completing clinical studies, where staff may bulk export every interpreted ECG. The archived box is a listing of every recording that was interpreted and confirmed (the final interpretation).
4. Select any ECG preview to open a full 30-second rhythm strip.

KardiaPro

ECG Inbox

Personal Inbox 6 2 Practice Inbox 45 Archived 3

ADD PATIENT

LAST REFRESHED 03:46 PM

	DATE	PATIENT	ANALYSIS	ANALYSIS BY	BPM - ECG PREVIEW
<input type="checkbox"/>	FILTER	FILTER	FILTER	FILTER	FILTER
<input type="checkbox"/>	10-15-2018 12:16 PM	Smith, Barbara 979646	Possible Atrial Fibrillation Kardia Instant Analysis	Morris, R	176 bpm
<input type="checkbox"/>	10-14-2018 04:03 PM	Auyeung, Jason 181251	Normal Kardia Instant Analysis	Morris, R	80 bpm
<input type="checkbox"/>	10-14-2018 02:36 PM	Bennett, Dominique 707643	Possible Atrial Fibrillation Kardia Instant Analysis	Yee, S	176 bpm
<input type="checkbox"/>	10-14-2018 01:13 PM	Lim, Kenneth 221664	Tachycardia Kardia Instant Analysis	Yee, S	176 bpm
<input type="checkbox"/>	10-14-2018 11:09 AM	Ingram, Vanessa 510734	Bradycardia Kardia Instant Analysis	Chang, D	40 bpm
<input type="checkbox"/>	10-14-2018 09:59 AM	Morales, Abner 286103	Possible Atrial Fibrillation Kardia Instant Analysis	Morris, R	176 bpm
<input type="checkbox"/>	10-13-2018 12:45 PM	Cloninger, Robyn 831422	Possible Atrial Fibrillation Kardia Instant Analysis	Yee, S	176 bpm
<input type="checkbox"/>	10-13-2018 11:08 AM	Harrison, Joshua 408553	Unclassified Kardia Instant Analysis	Morris, R	141 bpm

Reviewing ECGs (Cont.)

REVIEWING, INTERPRETING & SIGNING

1. Select "Add Interpretation" to manually input interpretation.

ECG Recording

Patients / Patient File / ECG Recording

MRN: R2D2 DOB: - Age: - yrs old Sex: Male Provider: Taylor, B Connection Plan: Morris AlV2 Connection Period: 01-27-2026 to (no end date)

ECG 6L 110 BPM Date Recorded: 2026-02-04 at 15:58

25 mm/s 10 mm/mV

Compare to previous ECG

Quick Actions

DOWNLOAD ECG

ECG Details

Analysis
Atrial Fibrillation
Heart Rate: 110 BPM
Device: 6L
Date Recorded: 2026-02-04 at 15:58

Measurements

QT Analysis: Algorithm Result
QT: 296 ms QTcF: 362 ms
RR: 545 ms QTcB: 401 ms

Patient Provided Data

Notes: -

Interpretation History

Atrial Fibrillation
Reviewed by: Sean Warren
Date Reviewed: 2026-02-16 at 15:26
Provider Note:
Escalate
Escalate
Confirmed By Warren, S 2026-02-16 at 15:26

Tachycardia
Reviewed by: Kardia Advanced Determination
Date Reviewed: 2026-02-04 at 15:58

You've reached the end of the list.

Reviewing ECGs (Cont.)

REVIEWING, INTERPRETING & SIGNING

1. Type in box or scroll to search for an interpretation. Time saving tip: You can easily search for "Ventricular Premature Complex" by typing "PVC" in the search bar. Additionally, you can select more than one interpretation. The interpretations selected will appear in the "added interpretations" section.
2. Optional: Type additional comments or notes.
3. Select "Save" to automatically e-sign and save your interpretation.

Reviewing ECGs (Cont.)

1. An interpretation added manually can be deleted before clicking "Confirm".

The screenshot displays the ECG Recording interface. At the top, it shows the patient's MRN (R2D2), age, sex (Male), provider (Taylor, B), and connection plan (Morris AlV2). The ECG recording itself shows a 12-lead rhythm (I, II, III, aVR, aVL, aVF) with a heart rate of 110 BPM. Below the ECG, there are quick actions including a 'DOWNLOAD ECG' button. The 'ECG Details' section lists the analysis as 'Atrial Fibrillation', heart rate as 110 BPM, and device as 6L. The 'Measurements' section shows QT: 296 ms, QTcF: 362 ms, RR: 545 ms, and QTcB: 401 ms. The 'Interpretation History' section shows two entries: one for 'Atrial Fibrillation' reviewed by Sean Warren on 2026-02-16 at 15:26, and another for 'Tachycardia' reviewed by Kardia Advanced Determination on 2026-02-04 at 15:58. A 'DELETE' button is visible next to the Atrial Fibrillation entry. The 'Patient Provided Data' section is currently empty.

Reviewing ECGs (Cont.)

OPTIONS FOR OVERREAD ECG

1. Select "Remove from Inbox," interpretation is saved in patient's file and added to summary report.
OR
2. Select "Reassign" to send rhythm strip to a different Inbox for review.
OR
3. Select "Confirm," interpretation is archived, saved in patient's file and added to summary report. By confirming, you will no longer be able to add or modify interpretations to this ECG. Confirmed ECGs will be listed in the archived folder. Option to bulk export confirmed ECGs from the archived folder.

OTHER ACTIONS

4. Export an individual ECG to a PDF file in one click.
5. Jump to a patient's file to view longitudinal ECG, blood pressure and weight data. Then easily use your browser back button to return to interpreting the ECG recording.

ECG Recording

Patients / Patient File / ECG Recording

MRN R2D2 DOB - Age - yrs old Sex Male Provider Taylor, B Connection Plan Morris AlV2 Connection Period 01-27-2026 to (no end date)

ECG 6L 110 BPM Date Recorded: 2026-02-04 at 15:58

25 mm/s 10 mm/mV

Compare to previous ECG

Quick Actions

DOWNLOAD ECG

ECG Details

Analysis
Atrial Fibrillation
Heart Rate: 110 BPM
Device: 6L
Date Recorded: 2026-02-04 at 15:58

Measurements

QT Analysis: Algorithm Result
QT: 296 ms QTcF: 362 ms
RR: 545 ms QTcB: 401 ms

Patient Provided Data

Notes: -

Interpretation History

Atrial Fibrillation
Reviewed by: Sean Warren
Date Reviewed: 2026-02-16 at 15:26
Provider Note:
Escalate
DELETE
Confirmed By Warren, S 2026-02-16 at 15:26

Tachycardia
Reviewed by: Kardia Advanced Determination
Date Reviewed: 2026-02-04 at 15:58
You've reached the end of the list.

Reviewing a Patient File

The patient directory is used to search for a specific patient. Once in a patient file, you'll have access to patients longitudinal data. From the patients file you'll also see notes, connection history, summary reports, and ECG, blood pressure and weight data.

1. Select "Patients."
2. To quickly find a specific patient, select "Filter" under the Patient column and type first name, last name or medical record number.
3. Select any row to open patient's file.

KardiaPro

Patient Directory ADD PATIENT LAST REFRESHED 02:03 PM

PATIENT	DOB	ORDERING PROVIDER	CONNECTION	CONNECTION PLAN	BILLING CODE
Barbara	-19-1958	Yee, S	Not Ordered	1 Year Connection Only	
Smith, Barbara 973646	-16-1963	Morris, R	Connected	KardiaMobile 6L + 1 Year Connection	CPT 99091
Chan, Barbara 522689	-14-1949	Pham, C	Connected	KardiaMobile 6L + BP Cuff + 1 Year Connection	CPT 99091
	-09-1963	Morris, R	Connected	KardiaMobile 6L + 1 Year Connection	CPT 99091
Banks, Andrew 758877	11-01-1952	Chang, D	Pending	KardiaMobile 6L + 1 Year Connection	
Bennett, Dominique 767643	07-01-1962	Yee, S	Pending	KardiaMobile 6L + 1 Year Connection	
Bolt, Tristan 949118	02-10-1960	Morris, R	Connected	KardiaMobile 6L + 1 Year Connection	CPT 99091
Chan, Barbara 522689	05-16-1954	Pham, C	Connected	KardiaMobile 6L + 1 Year Connection	CPT 99091

Reviewing a Patient File (Cont.)

1. To return to the full list of patients, select “Filter” again, and press the “x” to remove filter.

The screenshot displays the KardiaPro Patient Directory interface. On the left is a sidebar with navigation options: Dashboard, Patients, ECG Inbox, Reports, Settings, and Sign Out. The main area shows a table of patients with columns for Patient, DOB, Ordering Provider, Connection, Connection Plan, and Billing Code. A filter overlay is active over the Patient column, showing a search for 'Smith, Barbara' with a red '1' indicating the filter removal step. The table contains one visible entry for Barbara Smith.

PATIENT	DOB	ORDERING PROVIDER	CONNECTION	CONNECTION PLAN	BILLING CODE
Smith, Barbara 979646	3/48	Morris, R	Connected	KardiaMobile 6L + BP Cuff + 1 Year Connection	CPT 99091

Reviewing a Patient File (Cont.)

1. Sort patients by name, DOB, ordering provider, connection status, and connection plan, by simply clicking on the column header.
2. Select any row to open the patient's file.

KardiaPro

Patient Directory ADD PATIENT LAST REFRESHED 02:03 PM

PATIENT	DOB	ORDERING PROVIDER	CONNECTION	CONNECTION PLAN	BILLING CODE
<input type="text" value="FILTER"/>	<input type="text" value="FILTER"/>				
Aarons, Kyle 947835	03-19-1958	Yee, S	Not Ordered	1 Year Connection Only	
Allen, Kendrick 626348	07-16-1963	Morris, R	Connected	KardiaMobile 6L + 1 Year Connection	CPT 99091
Andersen, Leslie 484831	03-14-1949	Pham, C	Connected	KardiaMobile 6L + BP Cuff + 1 Year Connection	CPT 99091
Auyeung, Jason 181251	12-09-1963	Morris, R	Connected	KardiaMobile 6L + 1 Year Connection	CPT 99091
Banks, Andrew 758877	11-01-1952	Chang, D	Pending	KardiaMobile 6L + 1 Year Connection	
Bennett, Dominique 787643	07-01-1962	Yee, S	Pending	KardiaMobile 6L + 1 Year Connection	
Bolt, Tristan 949118	02-10-1960	Morris, R	Connected	KardiaMobile 6L + 1 Year Connection	CPT 99091
Chan, Barbara 522689	05-16-1954	Pham, C	Connected	KardiaMobile 6L + 1 Year Connection	CPT 99091
Chang, Austin 814720	06-01-1943	Yee, S	Connected	1 Year Connection Only	CPT 99091
Chin, Yvonne 529394	03-28-1953	Pham, C	Connected	KardiaMobile 6L + 1 Year Connection	CPT 99091
Cloninger, Robyn 831422	08-11-1940	Yee, S	Connected	KardiaMobile 6L + BP Cuff + 1 Year Connection	CPT 99091

Reviewing a Patient File (Cont.)

OVERVIEW: CONNECTED PATIENTS

1. From the "Overview" tab you can manage patient connection plan, modify billing codes, generate a new 12-letter referral code and manage inbox filters.
2. Patient connection status: Connected
3. For connected patients, you'll see the connection period.
4. You can modify any of the fields in this box by selecting the pencil icon to edit.
5. Primary Indication: This is populated from a library of ICD-10 diagnosis codes.
6. Billing Code: This must be selected to auto-generate summary reports.

The screenshot displays the KardiaPro interface for a patient named Barbara Smith, 70 years old, with MRN 979646. The page is titled "Overview" and includes a navigation menu on the left with options: Dashboard, Patients, ECG Inbox, Reports, Settings, and Sign Out. The main content area shows the following details:

- Connection Status:** Connected (Callout 2)
- Order Information:**
 - CONNECTION PLAN:** KardiaMobile 6L + BP Cuff + 1 Year Connection (Callout 4)
 - DURATION:** 1 year (Callout 3)
 - CONNECTION PERIOD:** 09-01-18 to 08-31-19 (Callout 3)
 - PRIMARY INDICATION:** Paroxysmal Atrial Fibrillation (Callout 5)
 - BILLING CODE:** CPT 99091 (Callout 6)
 - ORDER NUMBER:** 2238756
 - WHICH RECORDINGS TO FLAG:** All non-normal recordings
 - FLAGGED RECORDINGS GO TO:** Practice Inbox

At the bottom of the order information section, there are buttons for "REVOKE" and "SAVE". The top right corner features a "DOWNLOAD REPORT" button and an "ADD PATIENT" button. The last refreshed time is 02:29 PM.

Reviewing a Patient File (Cont.)

OVERVIEW: CONNECTED PATIENTS, CONTINUED

- Which Recordings to Flag: Modify at any time which recordings are flagged to for review.
 - If you'd like to customize the inbox settings for a specific patient, select the pencil icon next to "Which Recordings to Flag."
 - For example: If a patient has been diagnosed with AFib and you only want to see recordings with HR over 140, select "Custom" and you can designate this here. This allows you to set patient specific settings in order to flag actionable data.
- Revoke: Select to manually disconnect a patient from KardiaPro. You will then be asked to confirm that you want to disconnect the patient from the portal.

KardiaPro Barbara Smith 70 yrs old MRN 979646 [DOWNLOAD REPORT](#) [ADD PATIENT](#)

Overview Profile Reports ECG Blood Pressure Weight LAST REFRESHED 02:29 PM

Robert Morris
MORRIS CARDIOLOGY

- Dashboard
- Patients**
- ECG Inbox
- Reports
- Settings
- Sign Out

Connection Status

● Connected

Order Information

CONNECTION PLAN	DURATION	CONNECTION PERIOD
KardiaMobile 6L + BP Cuff + 1 Year Connection	1 year	09-01-18 to 08-31-19

PRIMARY INDICATION Paroxysmal Atrial Fibrillation	BILLING CODE CPT 99091	ORDER NUMBER 2238756
WHICH RECORDINGS TO FLAG All non-normal recordings	FLAGGED RECORDINGS GO TO Practice Inbox	

7 **8**

[REVOKE](#) [SAVE](#)

Reviewing a Patient File (Cont.)

OVERVIEW: PENDING PATIENTS

1. For pending patients, or those who have not yet connected to the KardiaPro portal, you'll see the 12-letter referral code. The referral code is good for 90 days. If the patient does not connect in 90 days the code will expire and no longer appear on this page. The page status will change from "Pending" to "Not ordered." If the patient would like to connect after the code expiration, simply order a new connection plan, which will issue a new referral code.

The screenshot shows the KardiaPro interface for a patient named Barbara Smith, 70 years old, with MRN 979646. The interface is divided into a left sidebar and a main content area. The sidebar includes a user profile for Robert Morris, Morris Cardiology, and navigation options: Dashboard, Patients, ECG Inbox, Reports, Settings, and Sign Out. The main content area displays the patient's profile at the top, followed by a navigation menu with 'Order' selected. Below this, the 'Connection Status' section shows a 'PENDING' status with a red notification badge '1' and a message: 'A Kardia order was sent to this patient Today at 2:22 pm. the patient has not connected to your practice using the below code. REFERRAL CODE ABCD-EFGH-IJKL'. The 'Order Information' section contains a table with the following details:

CONNECTION PLAN		DURATION
KardiaMobile 6L + BP Cuff + 1 Year Connection		11/1/18-10/31/19

PRIMARY INDICATION	BILLING CODE	EHR ORDER NUMBER
Paroxysmal atrial fibrillation	CPT 99091	00224423

WHICH RECORDINGS TO FLAG	FLAGGED RECORDINGS GO TO
All non-normal recordings	Practice Inbox

At the bottom of the main content area, there are two buttons: 'REVOKE' and 'SAVE'.

Reviewing a Patient File (Cont.)

OVERVIEW: NOT CONNECTED

1. A patient status is either connected, pending or not connected. If the patient is not connected (example: the patient completed a connection period and is no longer connected), you'll see the order form appear on the first tab.
2. Here you can order a new connection plan and create a new referral code.

The screenshot displays the KardiaPro interface for a patient named Barbara Smith, 70 years old, with MRN 979646. The 'Order' tab is active, indicated by a red '1' and a red '2' on the 'Connection Plan' form. The interface includes a sidebar with navigation options: Dashboard, Patients, ECG Inbox, Reports, Settings, and Sign Out. The main content area is divided into three sections: 'Connection Plan' (STEP 1), 'Reports & Billing' (STEP 2), and 'Inbox Settings' (STEP 3). The 'Connection Plan' section has a dropdown menu for 'Select a Connection Plan' and a text input for 'Enter Patient Email Address', both marked as 'Required'. The 'Reports & Billing' section has dropdowns for 'Select ICD-10 Code' and 'Select Billing Code', and a text input for 'Enter EHR Order Number'. The 'Inbox Settings' section has dropdowns for 'Which Recordings to Flag' and 'Flagged Recordings Go To'. At the bottom right, there is an 'ORDER' button with a checkmark icon. The top right corner features 'DOWNLOAD REPORT' and 'ADD PATIENT' buttons, and a 'LAST REFRESHED' timestamp of 10:38 AM.

Reviewing a Patient File (Cont.)

1. "Profile" tab of Patient File.
2. Patient demographic information, all fields are modifiable by selecting the pencil icon.
3. Access and modify notes section in concert with key points regarding patient care plan.
4. Scroll to see KardiaPro connection history.

KardiaPro Barbara Smith 70 yrs old MRN 979646 [DOWNLOAD REPORT](#) [ADD PATIENT](#)

Overview **Profile** Reports ECG Blood Pressure Weight LAST REFRESHED 02:29 PM

Patient Information

2 **Barbara** **Smith**

979646 **barbara@gmail.com** **Female**

08-29-1948 **Morris, Robert** **312-555-7767**

3 **Initial dx of paroxysmal AF Jan 2018 and had cardioversion. Patient still presenting w/intermittent AF episodes. Started univac for BP control in July 2018.**

[SAVE](#)

Connection History

4

START DATE	END DATE	CONNECTION PLAN	ICD10 CODE	BILLING CODE	ORDER NUMBER
09-01-2018	08-31-2019	KardiaMobile 6L + BP Cuff + 1 Year Connection	Paroxysmal AF	CPT 99091	2238756

Reviewing a Patient File (Cont.)


1. "Reports" tab of Patient File.
2. Review history of all auto-generated summary reports for this patient.
3. See which reports have e-signed and exported.
4. Note the workflow for reviewing, interpreting,
5. e-signing and exporting reports will be completed through the Reports tab, on the left navigation pane. Historical reports specific to the patient can then be accessed in the patient file in the Reports tab.

The screenshot displays the KardiaPro interface for a patient named Barbara Smith (70 yrs old, MRN 979646). The 'Reports' tab is selected, showing a table of reports. The left navigation pane includes options for Dashboard, Patients, ECG Inbox, Reports (highlighted with a red circle 4), Settings, and Sign Out. The patient's name and MRN are displayed at the top, along with a 'DOWNLOAD REPORT' button and an 'ADD PATIENT' button. The table below shows a report from 09-30-2018, ordered by Robert Morris, with a connection plan of 'KardiaMobile 6L + BP Cuff + 1 Year Connection', billing code 'CPT 99091', and both 'E-SIGNED' and 'EXPORTED' status set to 'Yes'. Red circles 1, 2, and 3 highlight the 'Reports' tab, the report row, and the 'E-SIGNED' and 'EXPORTED' columns respectively.

DATE	ORDERING PROVIDER	CONNECTION PLAN	BILLING CODE	E-SIGNED	EXPORTED	
<input type="checkbox"/>	<input type="text" value="FILTER"/>	<input type="text" value="FILTER"/>	<input type="text" value="FILTER"/>	<input type="text" value="FILTER"/>	<input type="text" value="FILTER"/>	
<input type="checkbox"/>	09-30-2018	Morris, Robert	KardiaMobile 6L + BP Cuff + 1 Year Connection	CPT 99091	Yes	Yes

Reviewing a Patient File (Cont.)

1. "ECG" tab of Patient File.
2. Use the scroll bar under the ECG graph to quickly scroll through months of ECGs to see visual representation for how frequently the patient is in and out of sinus rhythm. Tap on a dot in the graph to bring the ECG recordings to the top of the list.
3. View all ECGs this patient has recorded.
4. Open any ECG to apply interpretations. (see page 11 for further instructions)
5. Note the workflow for reviewing, interpreting, and e-signing flagged ECG's will be completed through the ECG Inbox, on the left navigation pane. Regardless of which recordings have been flagged for review, a copy of all ECG recordings for the patient will be in the patient's file in the ECG tab.

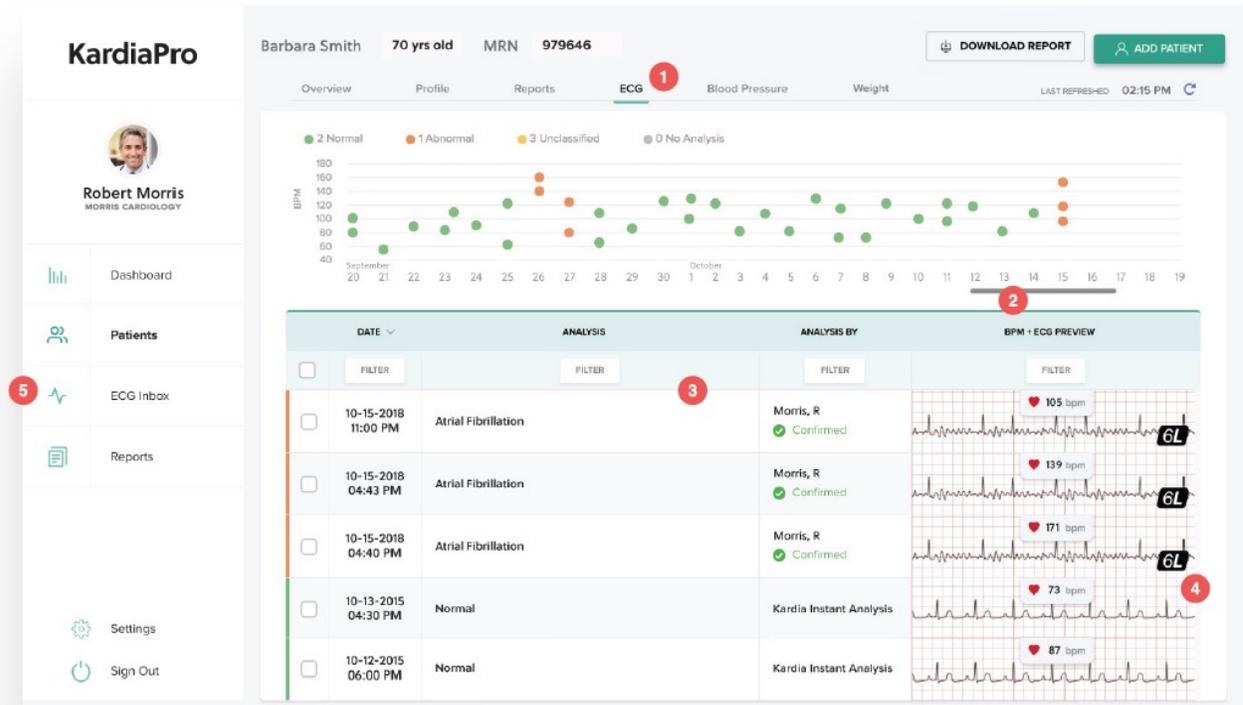

Reviewing Blood Pressure Data


KardiaPro gives you deeper insight of your patient's health by allowing you to easily spot trends and adjust medications as needed.

Blood pressure recordings are captured in the Kardia app by either:

1. Manual Entry: Patient manually enters blood pressure into their Kardia app using any blood pressure monitor.
OR
2. Omron Bluetooth technology: Patient uses an Omron Bluetooth enabled monitor to electronically transfer BP recordings into their Kardia app.


All blood pressure recordings will be available for review in the KardiaPro patient file under the blood pressure tab.

1. "Blood Pressure" tab of Patient File.
2. Filter and sort data by recording date.
3. Source will tell you if the recording was captured using a "Connected Device" or "Manual Entry."

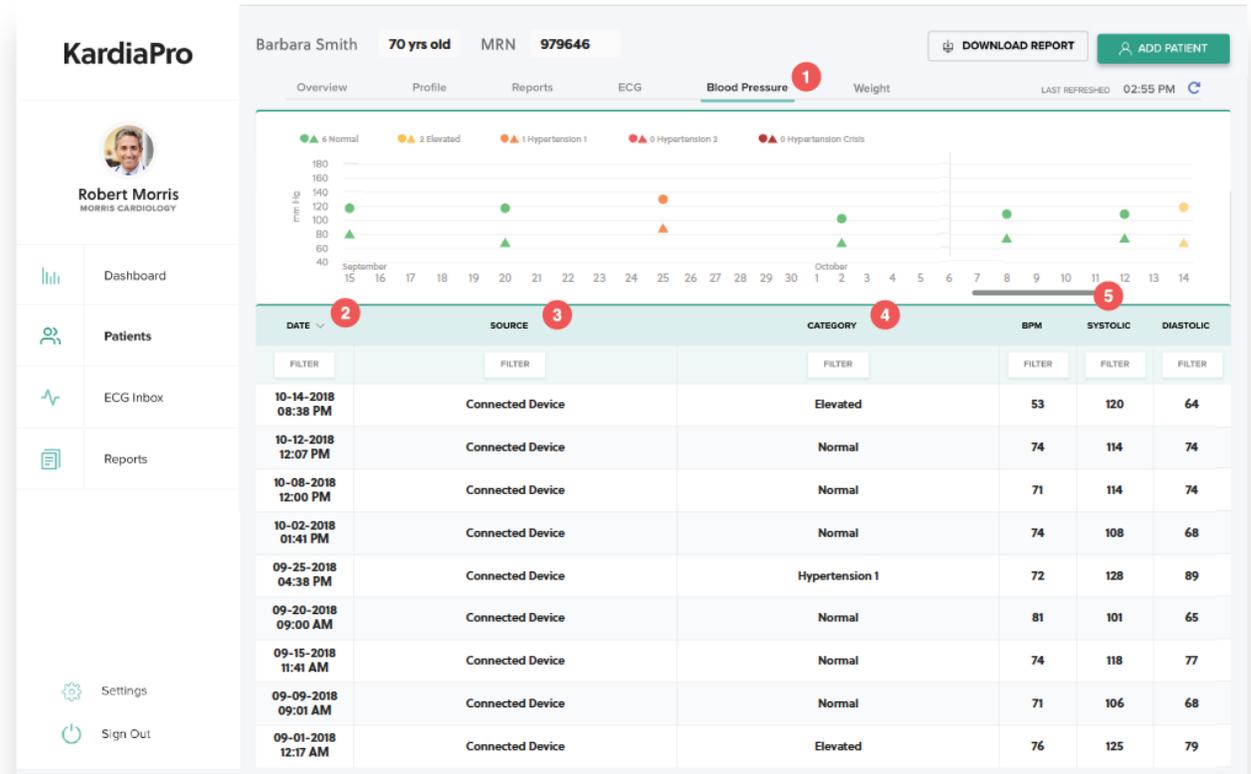

Reviewing Blood Pressure Data (Cont.)

4. Category is defined by the AHA blood pressure categories.
5. Filter data by selecting a range of Beats Per Minute (BPM), Systolic or Diastolic values.

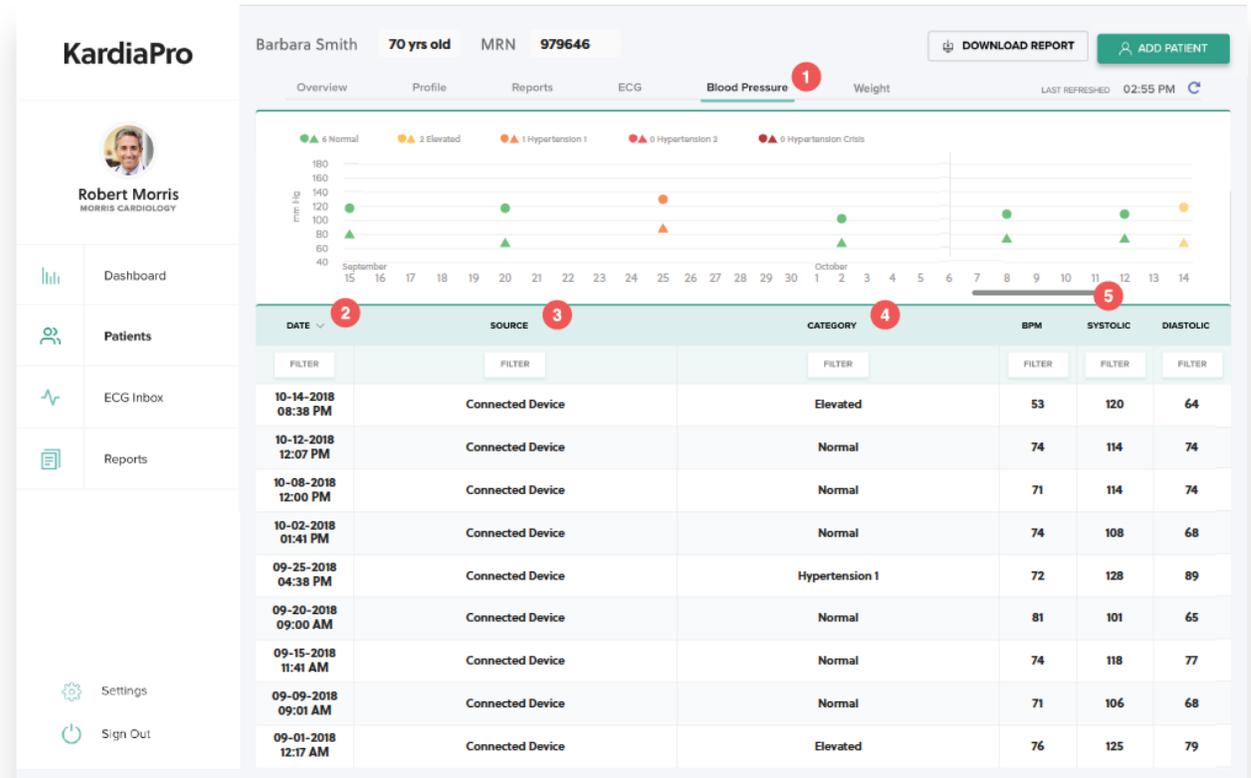

Reviewing Reports

Reports will be auto-generated for all patients assigned a billing code, as designated within the patient file overview page. Reports will be generated per the requirements of the specific billing code, i.e. auto-generation of a report every 30-days or every calendar month.

At a glance you'll see which reports have been e-signed and exported.

TO E-SIGN REPORTS

1. Select "Reports" from the left navigation pane.
2. Filter by the ordering provider.
3. Filter e-signed and select "No."
4. Open the report to review, interpret and add e-signature (see next page for details).

DATE	PATIENT	ORDERING PROVIDER	CONNECTION PLAN	BILLING CODE	E-SIGNED	EXPORTED
10-16-2018	Hughes, Anne 486526	Yee, S	KardiaMobile 6L + BP Cuff...	CPT 99091	No	No
09-30-2018	Smith, Barbara 979646	Morris, R	KardiaMobile 6L + 1 Year...	CPT 99091	No	No
09-25-2018	Allen, Kendrick 486553	Morris, R	1 Year Connection Only	CPT 99091	No	No
09-09-2018	Harrison, Joshua 488553	Yee, S	KardiaMobile 6L + 1 Year...	CPT 99091	No	No
08-09-2018	Morales, Abner 286183	Chang, D	KardiaMobile 6L + 1 Year...	CPT 99091	Yes	Yes
08-30-2018	Bolt, Tristan 949118	Yee, S	1 Year Connection Only	CPT 99091	Yes	Yes
08-30-2018	Nichols, Courtney 889675	Morris, R	1 Year Connection Only	CPT 99091	Yes	Yes
08-30-2018	Howell, Sam 626348	Pham, C	KardiaMobile 6L + 1 Year...	CPT 99091	Yes	Yes
08-30-2018	Ingram, Vanessa 510734	Morris, R	KardiaMobile 6L + 1 Year...	CPT 99091	Yes	Yes
08-30-2018	Laird, Leila 480428	Chang, D	1 Year Connection Only	CPT 99091	Yes	Yes

Reviewing Reports (Cont.)

TO E-SIGN REPORTS

1. Reports have several pages. Scroll up and down to see all of the pages of this patient file. The user can E-sign when viewing any page on the report.
2. The report will include a summary of all data captured in Kardia over the time period specified for the billing code selected.
3. Select "E-Sign" to add comments and e-signature with date.

KardiaPro

Report

E-SIGN

ADD PATIENT

View Patient File

LAST REFRESHED: 03:11 PM

KardiaPro Summary Report

Report Period: Sep 1, 2018 to Sep 30, 2018

Sample Patient MRN: 979646

DOB	Aug 29, 1948 (70 yrs)	Billing Code	CPT 99091
Sex	Female	Indication	Paroxysmal Atrial Fibrillation
Provider	Robert Morris	Connected	Sep 01, 2018 (EHR Order # 00224423)

EKGs

39 recordings

Legend: Normal (Green Circle), Non-normal (Orange Triangle), Unclassified (Yellow Square), No Analysis (Grey Circle)

200 BPM

150

100

50

1 2 3 4 5 6 7 8 9 10 11 12 13 14 15 16 17 18 19 20 21 22 23 24 25 26 27 28 29 30

September

Blood Pressure

26 recordings

Legend: Normal (Green Circle), Elevated (Yellow Circle), HTN 1 (Orange Circle), HTN 2 (Red Circle), HTN Crisis (Dark Red Circle)

180 mmHg

150

100

75

50

1 2 3 4 5 6 7 8 9 10 11 12 13 14 15 16 17 18 19 20 21 22 23 24 25 26 27 28 29 30

September

By time of day

120 mmHg

110

100

90

80

By category

180+

140

130

120

Systolic

Provider Comments

Reviewing Reports (Cont.)


TO BULK EXPORT REPORTS OR DOWNLOAD A BILLING REPORT CSV FILE

1. Filter e-signed and select "Yes."
2. Filter exported and select "No."
3. Check the left hand column for the reports you want to export or download.
4. To bulk export a PDF file of each report: Select "Export Patient Report" and then check your email for a secure link to a zip file of summary reports.
- OR
5. To download a billing report CSV file: Select "Download CSV" and a file will be immediately downloaded for you to open.

BILLING REPORT CSV FILE

The details included in the CSV file are:

- Report Start Date
- Report End Date
- Patient Name
- Patient MRN
- Patient DOB
- Ordering Provider Name
- ICD-10 Code
- Billing Code
- E-signed
- Exported

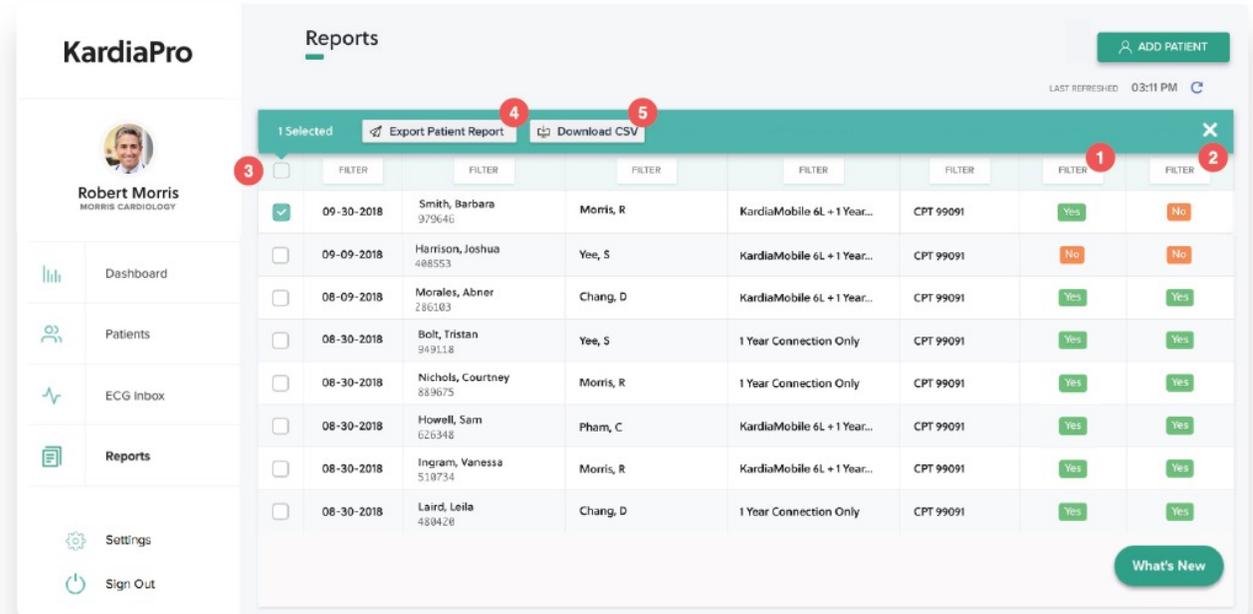

KardiaPro Multi-site Feature

Accessing multiple practices through one KardiaPro instance.

1. Once the Customer Support team has enabled the multi-site feature for a user, the user will be able to access all practices mapped through them via this drop-down feature.
2. After the practice has been selected from the drop-down menu, the name of the practice will be displayed
3. This is not a default offering. It can be only available once the customer support team has enabled it.

The screenshot shows the KardiaPro interface. On the left, a user profile for 'qa+usnov25su5@alivecor.c...' is shown with a 'Practice' dropdown menu. The dropdown menu is open, showing three options: 'KAI v2 test' (highlighted with a red circle and '1'), 'KardiaStation_TestPractice', and 'KiwiTeam1'. Below the menu are icons for 'ECG Inbox', 'Reports', and 'Support'. At the bottom of the sidebar are 'Settings' and 'Sign Out' options. The main dashboard area is titled 'Dashboard' and 'Fulfillment Dashboard'. It features a 'At a Glance' section with two ECG Inboxes: 'My ECG Inbox' (2 Incoming, 0 Needs Review, 0 Completed) and 'Practice ECG Inbox' (5584 Incoming, 64 Needs Review, 168 Completed). Below this is a 'Patient Connections' section with four categories: 187 Completed, 76 Pending, 14 (15 Days Left), and 4193 Not Ordered. The 'Recent ECG's' section contains a table with columns: Member, Date, Time, AI Determination, HR, Email, and Phone.

Member	Date	Time	AI Determination	HR	Email	Phone
Kpronov19	2024-11-29	04:26:18 PM	Normal Sinus Rhy...	97	qa+Kpronov19@alivecor.c...	+19995973537
Kpronov19	2024-11-29	03:50:04 PM	NOINTERPRETA...	95	qa+Kpronov19@alivecor.c...	+19995973537
uskanov29and2	2024-11-29	01:47:26 PM	Normal Sinus Rhy...	94	qa+uskanov29and2@alive...	+19886567888

User Settings


From Settings, all users can change their password.

For personnel designated as Account Administrators, you can add and remove members and modify Inbox default settings.

ALL KARDIAPRO USERS: UPDATE PASSWORD

1. Select "Settings."
2. "Update Password" tab of Settings.
3. Enter your Current Password.
4. Enter your New Password twice.
5. Select "Save."



Alternatively, if you don't know your password and you'd like to reset your password, you can go to kardiapro.com and click "forgot password." Enter your email. This will generate an email with a link to reset your password.

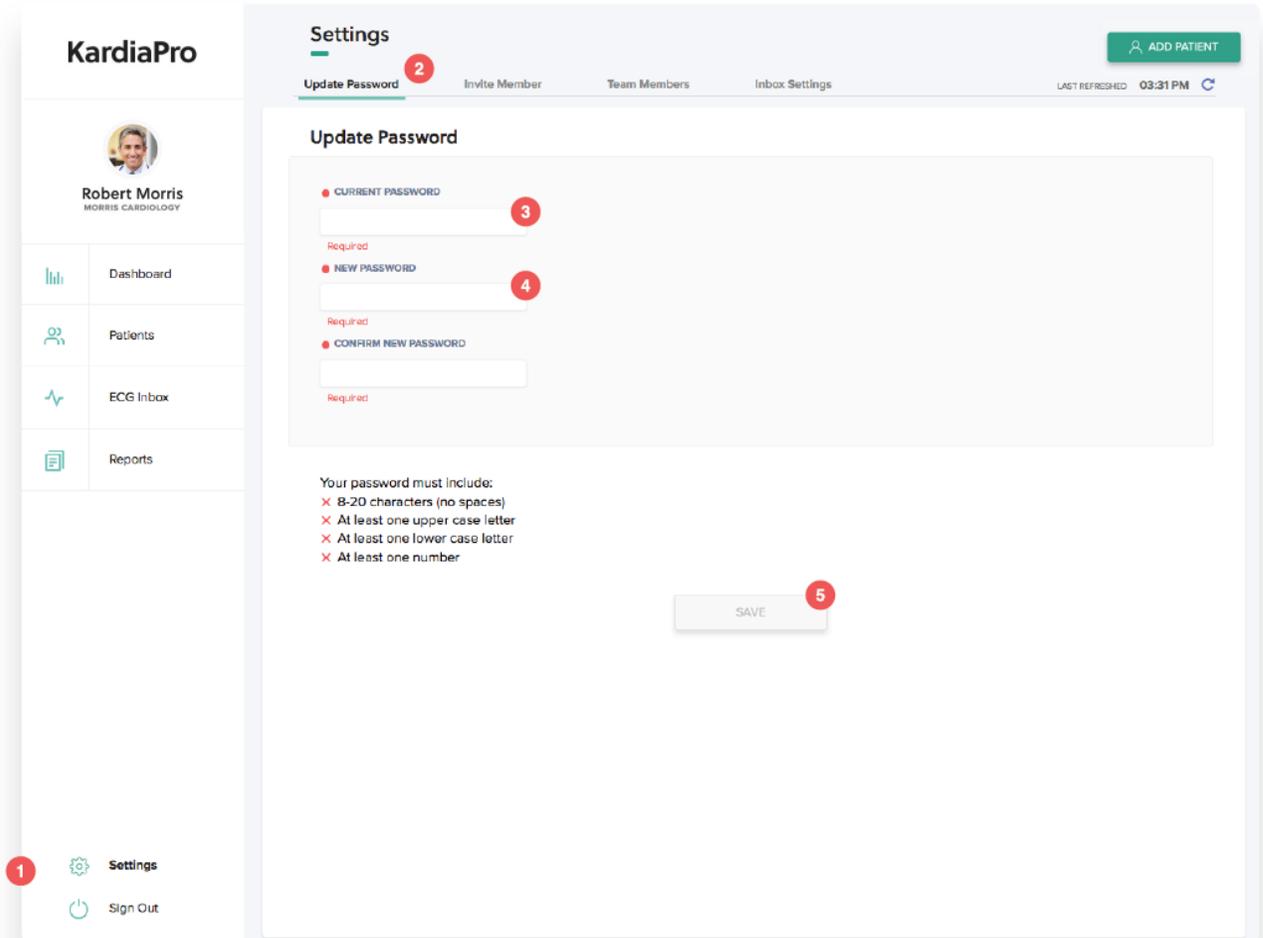

User Settings (Cont.)

ACCOUNT ADMINISTRATOR: INVITE MEMBER

1. Designate Member Roles by selecting the appropriate checkbox.
 - Administrator: Can manage team member accounts and Inbox settings.
 - Provider: provides overreads. Will be listed as an Ordering Provider when adding new patients.
 - Technician: Can record ECGs for patients using the AliveCor app.
 - Viewer: Has view-only access to the portal.
 - A user may hold multiple roles, except for Viewers, who can only be assigned the Viewer role exclusively.
2. Pending Invites can be revoked by checking the email and selecting "Revoke."

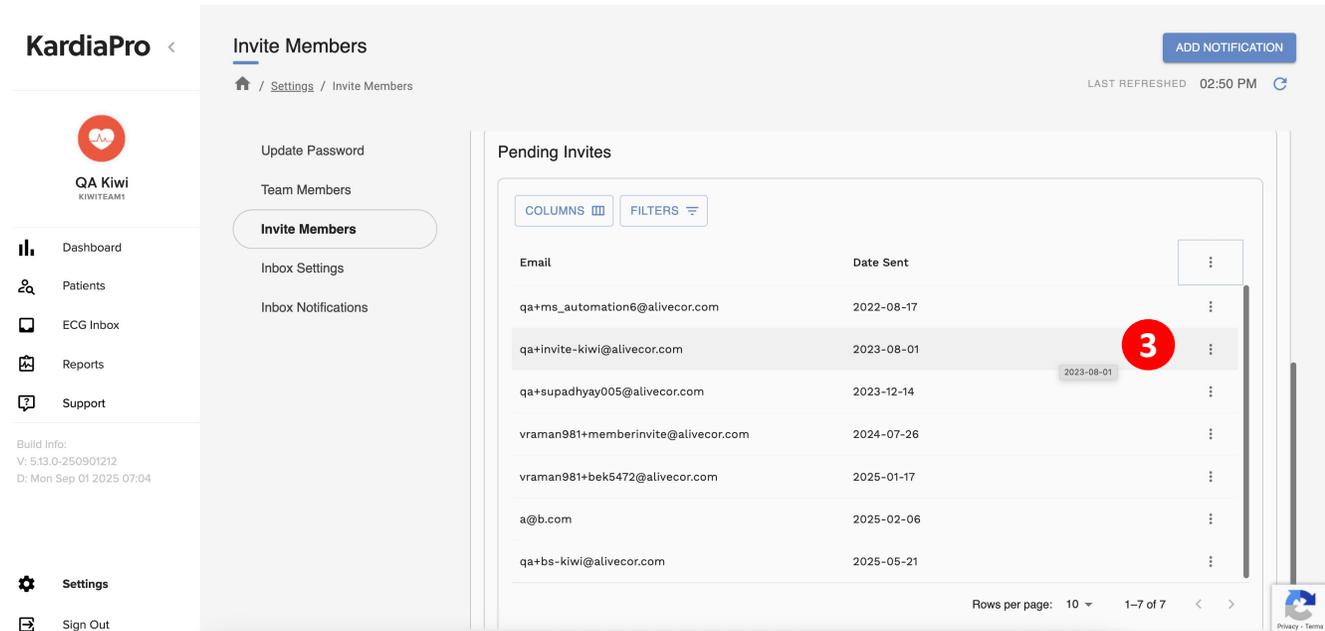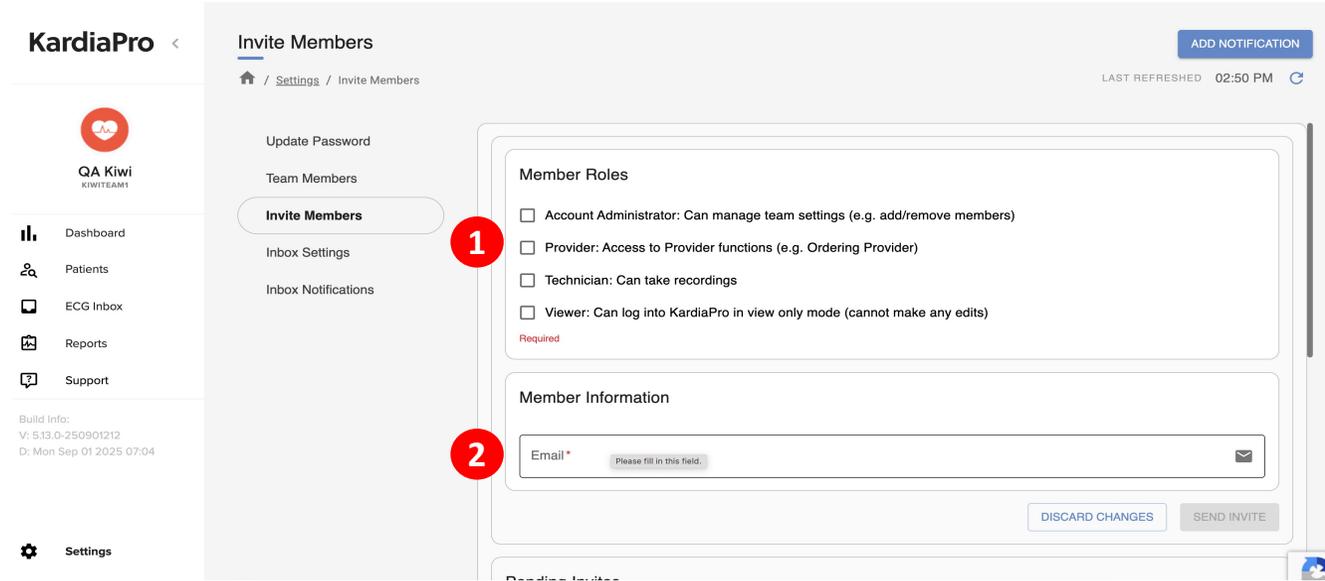

User Settings (Cont.)

ACCOUNT ADMINISTRATOR: MEMBER INFORMATION

1. "Team Members" tab of Settings.
2. Search by name or email by selecting "Filter."
3. Quickly filter or sort list by clicking at the relevant role.
4. You can open any team member information by selecting any row. (see next page for details)

KardiaPro <

Team Members

Update Password

Team Members 1

Invite Members

Inbox Settings

Inbox Notifications

ADD NOTIFICATION

LAST REFRESHED 02:59 PM

Team Member

COLUMNS FILTERS 2

Team Mem...	Email	Is Adm... 3	Is Provider	Is Viewer	Is Technician	Is Super... ↑
Alivecor1 V...	vkaushal288+kiwi2@alivec...	Yes	Yes	No	Yes	No
Dutt Rit	rdutta890@alivec.com	Yes	Yes	No	No	No
4 (T...	vkaushal288+invitetest@al...	No	No	No	No	No
Kaushal Vai...	vkaushal288+kiwitest1@ali...	Yes	Yes	No	Yes	No
		Yes	Yes	No	Yes	No

Rows per page: 10 1-10 of 63

Privacy - Terms

User Settings (Cont.)

ACCOUNT ADMINISTRATOR: MEMBER INFORMATION

1. Click to edit name, email address and assigned role.
2. Click to edit email reminder settings.
3. Click to delete user.

KardiaPro < **Team Member**

🏠 / Settings / Team Members / Team Member Information

QA Kiwi
KIWITEAM1

- Dashboard
- Patients
- ECG Inbox
- Reports
- Support

Build Info:
V: 5.13.0-250901212
D: Mon Sep 01 2025 07:04

Settings
Sign Out

Member Roles

- Account Administrator: Can manage team settings (e.g. add/remove members)
- Provider: Access to Provider functions (e.g. Ordering Provider)
- Technician: Can take recordings
- Viewer: Can log into KardiaPro in view only mode (cannot make any edits)

Member Information

First Name	Last Name	Email
Vaibhav	Kaushal (TEST)	vkaushal288+invitetest@alivecor.com

1 [EDIT MEMBER INFORMATION](#)

Email Reminder

Send an email reminder when ECGs and reports need attention.

Frequency of reminders: **Never**

2 [EDIT REMINDER](#)

3 [DELETE MEMBER](#)

Privacy

User Settings (Cont.)

ACCOUNT ADMINISTRATOR: INBOX SETTINGS

1. "Inbox Settings" tab of Settings.
2. Select the practice level default for which recordings to flag: All non-normal recordings, All Recordings, No Recordings or Custom.
3. Select "Save."

NOTE: The inbox settings tab sets the default that will appear when a user is adding a new patient. However you can always modify the "default inbox settings" for each patient upon patient enrollment or within the overview tab of the patient file.

The screenshot shows the KardiaPro interface. On the left is a sidebar with the user's name 'Robert Morris' and a list of navigation items: Dashboard, Patients, ECG Inbox, Reports, Settings, and Sign Out. The main area is titled 'Settings' and has several tabs: 'Update Password', 'Invite Member', 'Team Members', and 'Inbox Settings' (which is selected and marked with a red '1'). Below the tabs is a section for 'Team Inbox Settings'. It contains a dropdown menu titled 'WHICH RECORDINGS TO FLAG' (marked with a red '2') with the following options: 'All abnormal recordings', 'All recordings', 'All non-normal recordings', 'No recordings', and 'Custom'. Below the dropdown is a 'SAVE' button (marked with a red '3'). In the top right corner, there is an 'ADD PATIENT' button and a 'LAST REFRESHED 03:32 PM' indicator.